

Elite Bed Range Assembly

Quick Setup Guide

For persons who are familiar with the Elite Bed Range



WARNING: This Guide is only to be used in conjunction with the Instructions for Use, which must be read thoroughly before setting up the bed.



1 - Gather Components

- Make sure you have all components listed
- Clear space for assembling the Bed






2 - Assemble Deck

- Lay deck sections mattress side down
- Secure 1 Joining Prong in each section
- Push sections together and tighten

5 - Cable Routing

- Route head and foot end cables OVER the two red bars
- Fit Mains Cable strain relief
- Loop Handset & Mains cable through strain relief hooks on Control Unit

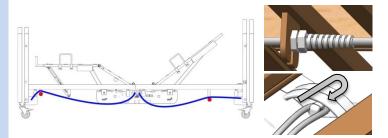



Connections;

BLANK PLUG

HEAD END

HANDSET


FOOT END

6 - Connect Cables

- Remove the cable cover screws & cover
- Connect the cables to the sockets.
- Replace Cover
- Check Emergency Batteries

3 - Fit Control Unit

- Remove covers from each end of the Unit
- Locate correctly over Head & Foot drive shafts
- Replace covers and lock in place

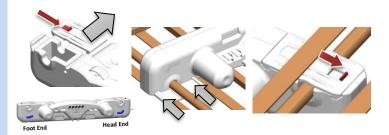

Head End Foot End

4 - Fit Bed Ends

- Identify Head & Foot ends
- Insert Bed End prongs into corresponding Bed Deck Tubes
- Tighten 2 handwheels at each end

7 - Final Checks

- Locate Bed in its final position
- Connect Mains power
- Check Bed movements correspond with handset buttons
- If mattress retainers are the removable type, ensure they are all correctly fitted
- Fit Mattress & apply all 4 brakes





8 - Side Rails (optional)

- Fit the Glide prongs through an endcap into a wooden Side Rail
- Depress lower button and slide glide up into rail
- Repeat on each bed corner post

Issue 2, Aug 2017 Page 1 of 2



Elite Bed Range Assembly

Quick Disassembly Guide

For persons who are familiar with the Elite Bed Range




WARNING: This Guide is only to be used in conjunction with the Instructions for Use, which must be read thoroughly before setting up the bed.



1 - Remove all Bedding

- Remove any fitted bedding
- Remove any accessories
- Remove Mattress

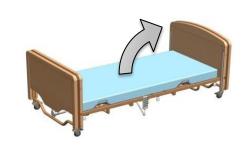




2 - Side Rails (if fitted)

- Raise Bed
- Lower all Side Rails
- Slightly lift lower Side Rail, & press the bottom Side Rail button
- Lower Side Rail Glide out of Bed End
- Repeat on all corners

5 - Remove Bed Ends

- Unlock Control Unit Covers
- If space permits, tip the bed frame on its side. & move the backrest to its raised position
- Loosen Handwheels & remove Bed Ends



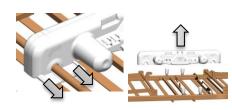

6 - Remove Control Unit

- Lay mattress platform flat, (mattress side down)
- Remove Control Unit
- Replace Control Unit Covers

3 - Mattress **Retainers & Position**

- Remove all 4 Mattress Retainers (if removable)
- Ensure you have good access to Control Box
- Lower Bed, disconnect from Mains & apply




4 - Disconnect Cables

- Unscrew Cable Cover
- Disconnect Handset, and Bed Ends
- Replace Cable cover and Screws

7 - Separate Deck

- Loosen 4 remaining handwheels
- Pull Deck sections apart
- Remove Deck Joining Bars

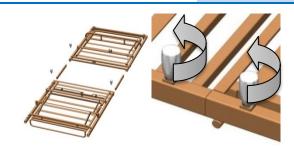



components individually



OR

Contact Renray for details of **Transport Brackets**

8 - Move or Store

- Screw all handwheels into deck
- Place Mattress Retainers, Joining Bars, Handset, Side Rail Caps & Glides and Instructions into a bag
- Zip-tie bag to Frame



Page 2 of 2 www.renrayhealthcare.com