


Habillage sommier haut

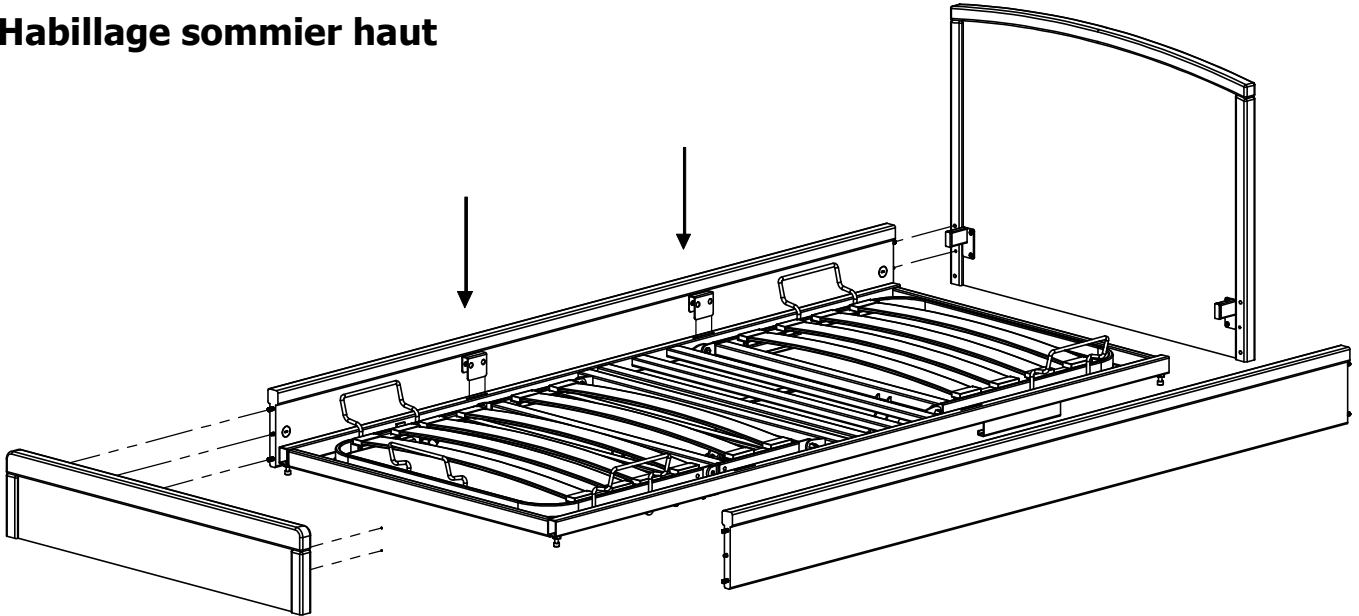

1

Install the mattress platform long boards in the slides located on each side of the mattress platform.

2

Screw in the metal brackets into the boards with the wood screws, in the visible holes.

3

Screw in the metal stud bolt in the head and footboards.

4



Place the head and footboards facing the sides of the long boards on each end of the bed and push in the metal stud bolts.

5

Screw in the eccentric rams a quarter-turn to lock in the frame.

6


Block the metal board brackets with the M8X20 screws and the allen wrench 5 mm.

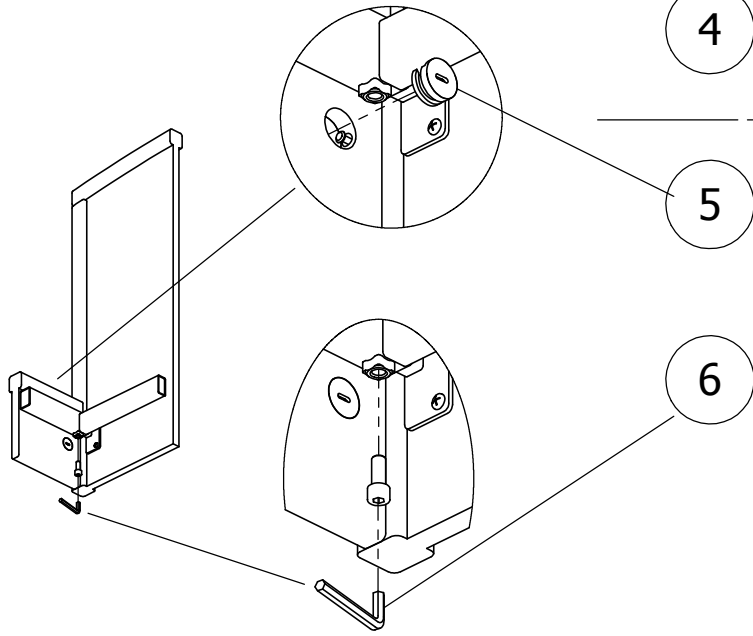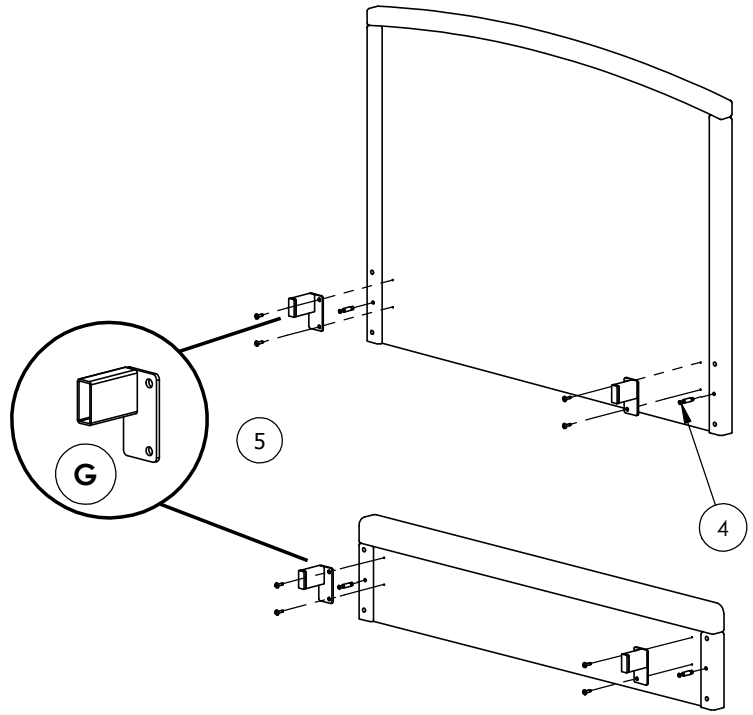

Warning: please read carefully the instructions before using the "lit-releveur®" bed

- the bed is not suitable for a child under 12 or for a person weighing more than 135kgs.
- the "lit-releveur®" should only be used on a flat, clear and horizontal floor.
- before you move the bed around, make sure you have unplugged it from the mains and rolled in the cable on the cable tidy.
- make sure the cables are plugged in the relevant sockets on the control box:

- 1.lock out function
- 2.handset
- 3.back rest actuator
- 4.variable height motor
- 5.legs rest actuator

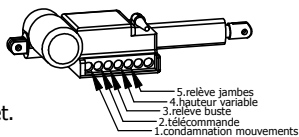

- keep children under 12 away when operating the bed up and down with the handset.
- make sure not to pinch, flatten, thread or knot the cables.
- some prevention maintenance must be scheduled and take place at least once a year throughout the bed lifetime, by a trained technician.
- you should disinfect the bed with a wet sponge and soap.
- unplug the bed for the mains when you disinfect the "lit-releveur®" bed.
- do not use acid, alkaline products or solvents such as acetone or cellulose diluents.
- the electric set up is IP 66 throughout:
- for this reason check that all connections to the central unit are effective
- please check that the earthing is safe

| Symptômes | Causes possibles | Solutions |
|--|--|---|
| The orange indicator for the mains on the handset is off | 1-the bed is not plugged to the mains 2-a fuse has blown 3-the central unit is faulty 4-the handset is faulty or its cord is torn off | 1-plug the bed to the mains 2-check both the mains cable and the handset cord 3-check the handset 4-replace the central unit |
| The orange indicator for the mains is on but the motor does not work. The relay in the central unit makes a "clinking" sound | 1-the motor connector is not properly plugged into the central unit 2-the motor is faulty 3-the central unit is faulty | 1-plug in the motor connector properly 2-replace the motor 3-replace the central unit |
| The orange indicator for the mains is on but the motor does not work. The relay in the central unit does not make any sound. | 1-the central unit is faulty 2-the handset is faulty 3-those functions are not allowed/locked | 1-replace the central unit 2-replace the handset 3-unlock those functions 4-push in the red « plug » |
| The central unit works properly except on one of the outputs | 1-check the connections 2-the handset is faulty 3-the central unit is faulty | 1-replace the handset 2-replace the central unit |
| The motor works but the drive shaft does not move | The pinion gear or the drive shaft is damaged | Replace the motor |
| The motor cannot lift the bed when the patient uses it | The motor is damaged | Replace the motor |
| The jack shaft does not move whilst the jack is turning | The safety nut has been removed | Replace the motor |

Electrical parts are warranted; the protection category is IP66. please do not attempt to open the motors: the warranty would not be valid anymore and the motors would not be waterproof anymore. Only staff from an authorised dealer can open and work on the electrical parts.



HMS-VILGO
243 rue Salvador Allende 59120 LOOS
e-mail : sav@hms-vilgo.com

HMS - VILGO

LIT-RELEVEUR®

Notice montage lit Euro 4000

dossier M09-02-01L edition 2009/02

Habillage châssis bas

